

Manual Daily Test

- First inspect the defibrillator for signs of damage.
 - It is not necessary to switch the AED on for the daily test!
 - Observe the display to the top right side of the AED, and perform the following checks.
1. The 'OK' symbol is visible.
 2. The battery and service indicator shows more than one bar, and does not show empty (no bars) or a spanner symbol.
 3. Check that there is one pack of in date 'Quik-Combo' pads preconnected to the AED, and that the pack of pads is sealed and in date.
 4. There should be one additional set of adult Quik-Combo pads.

New Weekly Check

- Remove battery and reinstall straight away

Physio-Control now part of Stryker
35 Great St Helens London, EC3A 6AP
Tel 0808 258 0094 /option 1

Battery and Service Indicator



Battery is fully charged



Order Replacement Battery



Battery is 25% charged or less
Replace immediately



Battery is exhausted
Replace immediately



Spanner indicates maintenance is required
urgently **Contact MESU**



www.physio-control.com

LIFEPAK® 1000 Defibrillator



stryker

LIFEPAK 1000

at a glance



The LP1000 AED performs an automatic self test at 3am every night – an alarm will sound and the spanner symbol will be displayed if a fault is detected.

Using a LIFEPAK 1000

for AED Defibrillation

Follow these 3 steps:

1. Apply QUIK-COMBO® REDI-PAK™ electrodes to patients chest
2. Press the **GREEN ON** button - Auto analyse will take place
3. If a shock is advised, press **RED SHOCK** button



Adult pads with Pre-connect



Paediatric attenuated pads. Resistor box governs down energy to safe levels for children >1yr